

Non WHO Reference Material NIBSC Anti-CMV IgG Serology Reagent NIBSC code: 25-B918 Instructions for use (Version 1.0, Dated 24/01/2025)

This material is not for in vitro diagnostic use

1. INTENDED USE

This reagent is intended for performance monitoring of immunoassays detecting IgG antibodies to CMV.

The reagent is expected to be detected by assays for CMV IgG, different batches of the same assay kit, and/or different assay conditions, may give different values with this reagent.

The reagent is not intended to be used for assay calibration purposes, or for in vitro diagnostic use.

2. CAUTION

This preparation is not for administration to humans or animals in the human food chain.

25/B918 has been prepared from a pool of anti—CMV IgG reactive defibrinated plasma donations, repeatedly reactive in commercial EIA kits . The reactive donations used to prepare 25/B918 were non-reactive for HBsAg, anti-HIV1/2 and anti-HCV using commercial EIA kits. The reactive donations were pooled and then diluted in a pool of defibrinated human plasma donations. These samples were non-reactive for HBsAg, anti-HCV and anti-HIV1/2, Anti-HTLV and Anti-Syphilis using commercial EIA kits. Bronidox[®] was added toa concentration of 0.05%(w/v) as a preservative. As with all materials of biological origin, this preparation should be regarded as potentially hazardous to health. It should be used and discarded according to your own laboratory's safety procedures. Such safety procedures should include the wearing of protective gloves and avoiding the generation of aerosols. Care should be exercised in opening ampoules or vials, to avoid cuts.

3. UNITAGE

There is no unitage assigned to this product.

4. CONTENTS

Country of origin of biological material: United Kingdom. Ready-to-use reagent REF 25/B918 1x7mL Blood Tubes Defibrinated Plasma 4mL Bronidox® (Sigma Aldrich) 0.05% (w/v)

5. STORAGE

Reagents are to be kept at 2-8°C upon receipt

o Reagents may be stored at 2-8°C until use by date

o Reagents should be divided into measured sub-aliquots of one use and

stored below -20°C to avoid freeze/thaw cycles. Use only until the expiry

dateon the label.

o When thawed for use, store at 2-8°C.

o Once thawed, use within one month and do not refreeze

o Ensure all containers are properly sealed to avoid drying out of reagent

National Institute for Biological Standards and Control, Potters Bar, Hertfordshire, EN6 3QG. T +44 (0)1707 641000, nibsc.org WHO International Laboratory for Biological Standards,

UK Official Medicines Control Laboratory

6. DIRECTIONS FOR OPENING

Vials have a screw cap; an internal stopper may also be present. The cap should be removed by turning anti-clockwise. Care should be taken to prevent loss of the contents. Please note: If a stopper is present on removal of the cap, the stopper should remain in the vial or be removed with the cap.

7. USE OF MATERIAL

- 1. Use of this reagent is to be restricted to trained laboratory staff only
- 2. Use suitable (latex/nitrile) gloves and eye/skin protection
- 3. Include reagent as a normal sample in routine work list
- 4. Allow reagent to reach room temperature before use
- 5. Plot reagent result on a QC chart to monitor performance

8. STABILITY (Add or amend as necessary)

Reference materials are held at NIBSC within assured, temperaturecontrolled storage facilities. Reference Materials should be stored on receipt as indicated on the label.

Stability studies are carried out to ensure stated stability.

9. REFERENCES

10. ACKNOWLEDGEMENTS

11. FURTHER INFORMATION

Further information can be obtained as follows; This material: enquiries@nibsc.org WHO Biological Standards: http://www.who.int/biologicals/en/ JCTLM Higher order reference materials: http://www.bipm.org/en/committees/jc/jctlm/ Derivation of International Units: http://www.nibsc.org/standardisation/international_standards.aspx Ordering standards from NIBSC: http://www.nibsc.org/products/ordering.aspx NIBSC Terms & Conditions: http://www.nibsc.org/terms_and_conditions.aspx NIBSC Results Reporting System: http://www.nibsc.org/products/rrs.aspx

12. CUSTOMER FEEDBACK

Customers are encouraged to provide feedback on the suitability or use of the material provided or other aspects of our service. Please send any comments to enquiries@nibsc.org

Alternatively, comments may also be addressed to $\ensuremath{\mathsf{qcru}}\xspace@nibsc.org$

13. CITATION

In all publications, including data sheets, in which this material is referenced, it is important that the preparation's title, its status, the NIBSC code number, and the name and address of NIBSC are cited and cited correctly.

14. MATERIAL SAFETY SHEET (Add or amend as necessary)



Classification in accordance with Directive 2000/54/EC, Regulation (EC) No 1272/2008: 1

Physical and Chemical properties				
Physical appearance: Corrosive: No				
Liquid			001103170.	
Stable: Yes		Oxidising:	No	
Hygroscopic: No		Irritant:	No	
Flammable: No		Handling:See caution, Section 2		
Other (specify):				
Toxicological properties				
Effects of inhalation:		Not established, avoid inhalation		
Effects of ingestion:		Not established, avoid ingestion		
Effects of skin		Not established, avoid contact with		
absorption:		skin		
Suggested First Aid				
Inhalation: Seek medical advice				
Ingestion: Seek medical advice				
Contact with	Wash with copious amounts of water. Seek			
eyes:	medical advice			
Contact with skin:	Wash thoroughly with water.			
Action on Spillage and Method of Disposal				
Spillage of ampoule contents should be taken up with absorbent material wetted with an appropriate disinfectant. Rinse area with				

an appropriate disinfectant followed by water.

Absorbent materials used to treat spillage should be treated as biological waste.

15. LIABILITY AND LOSS

In the event that this document is translated into another language, the English language version shall prevail in the event of any inconsistencies between the documents.

Unless expressly stated otherwise by NIBSC, NIBSC's Standard Terms and Conditions for the Supply of Materials (available at http://www.nibsc.org/About_Us/Terms_and_Conditions.aspx or upon request by the Recipient) ("Conditions") apply to the exclusion of all other terms and are hereby incorporated into this document by reference. The Recipient's attention is drawn in particular to the provisions of clause 11 of the Conditions.

16. INFORMATION FOR CUSTOMS USE ONLY

Country of origin for customs purposes*: United Kingdom * Defined as the country where the goods have been produced and/or sufficiently processed to be classed as originating from the country of supply, for example a change of state such as freezedrying.

Net weight: 7g

Toxicity Statement: Toxicity not assessed

Veterinary certificate or other statement if applicable.

Attached: No

National Institute for Biological Standards and Control, Potters Bar, Hertfordshire, EN6 3QG. T +44 (0)1707 641000, nibsc.org WHO International Laboratory for Biological Standards, UK Official Medicines Control Laboratory Medicines & Healthcare products Regulatory Agency



National Institute for Biological Standards and Control, Potters Bar, Hertfordshire, EN6 3QG. T +44 (0)1707 641000, nibsc.org WHO International Laboratory for Biological Standards, UK Official Medicines Control Laboratory