



**WHO International Standard
Haemiglobincyanide
NIBSC code: 98/708
Instructions for use
(Version 4.0, Dated 14/04/2008)**

1. INTENDED USE

A batch of material was prepared by Diagnostic Reagents, Thame, UK. Part of this batch became the EU Community Bureau of Reference, Certified Reference Material (CRM) 522. Another part of the same batch became the 6th International Standard for Haemiglobincyanide. With effect from 23rd October 1998, the National Institute for Biological Standards and Control (NIBSC), Potters Bar, UK is the custodian and distributor of this material. The standard has been assigned the NIBSC Code number 98/708.

2. CAUTION

This preparation is not for administration to humans or animals in the human food chain.

This preparation contains material of bovine origin. As with all materials of biological origin, this preparation should be regarded as potentially hazardous to health. It should be used and discarded according to your own laboratory's safety procedures. Such safety procedures should include the wearing of protective gloves and avoiding the generation of aerosols. Care should be exercised in opening ampoules or vials, to avoid cuts.

3. UNITAGE

49.8 micromoles/litre (0.803 grams/litre). For further information, see WHO BS 98.1886.

4. CONTENTS

Country of origin of biological material: United Kingdom.
Each ampoule contains 10ml of Haemiglobincyanide solution.

5. STORAGE

Store at +4°C. Do not freeze and thaw this material.

Please note: because of the inherent stability of lyophilized material, NIBSC may ship these materials at ambient temperature.

6. DIRECTIONS FOR OPENING

Care should be taken to prevent loss of the contents. Prior to opening the ampoules must be allowed to equilibrate at room temperature for at least one hour. After ensuring that the contents are entirely in the body of the ampoule, the snap top should be broken off. Dispense the required amount into a spectrometer cuvette for direct reading of absorbance at 540 nm. The material in the ampoule can be used for up to 8 hours if covered with an airtight seal and stored at +4°C prior to use, but must be discarded at the end of the working day.

7. USE OF MATERIAL

The International Council for Standardisation in Haematology and the World Health Organisation have recommended measurement of haemiglobincyanide, for determining the haemoglobin concentration in blood, in a spectrometer at 540 nm.

The preparation 98/708 is a dilute solution of haemiglobincyanide for direct reading in a spectrometer. It is intended to be used to calibrate the spectrometer at 540 nm and to check the validity of manufactured preparations of haemiglobincyanide which are used as secondary standards for internal quality control of spectrometers, photometers and blood cell counters used in routine haemoglobinometry.

It can also be used to prepare calibration graphs in order to convert absorbance measurements into haemoglobin concentration in g/l.

8. STABILITY

Reference materials are held at NIBSC within assured, temperature-controlled storage facilities. Reference Materials should be stored on receipt as indicated on the label.

It is the policy of WHO not to assign an expiry date to their international reference materials. They remain valid with the assigned potency and status until withdrawn or amended. NIBSC follows the policy of WHO with respect to its reference materials.

Users who have data supporting any deterioration in the characteristics of any reference preparation are encouraged to contact NIBSC.

9. REFERENCES

WHO BS 98.1886.

10. ACKNOWLEDGEMENTS

N/A

11. FURTHER INFORMATION

Further information can be obtained as follows;

This material: enquiries@nibsc.org

WHO Biological Standards:

<http://www.who.int/biologicals/en/>

JCTLM Higher order reference materials:

<http://www.bipm.org/en/committees/jc/jctlm/>

Derivation of International Units:

http://www.nibsc.org/standardisation/international_standards.aspx

Ordering standards from NIBSC:

<http://www.nibsc.org/products/ordering.aspx>

NIBSC Terms & Conditions:

http://www.nibsc.org/terms_and_conditions.aspx

12. CUSTOMER FEEDBACK

Customers are encouraged to provide feedback on the suitability or use of the material provided or other aspects of our service. Please send any comments to enquiries@nibsc.org

13. CITATION

In all publications, including data sheets, in which this material is referenced, it is important that the preparation's title, its status, the NIBSC code number, and the name and address of NIBSC are cited and cited correctly.

14. MATERIAL SAFETY SHEET

Classification in accordance with Directive 2000/54/EC, Regulation (EC) No 1272/2008: Not applicable or not classified

Physical and Chemical properties	
Physical appearance: Liquid	Corrosive: No
Stable: Yes	Oxidising: No
Hygroscopic: No	Irritant: No
Flammable: No	Handling: See caution, Section 2
Other (specify):	Contains cyanide
Toxicological properties	
Effects of inhalation:	Not established, avoid inhalation
Effects of ingestion:	Not established, avoid ingestion
Effects of skin absorption:	Not established, avoid contact with skin
Suggested First Aid	
Inhalation:	Seek medical advice



Ingestion:	Seek medical advice
Contact with eyes:	Wash with copious amounts of water. Seek medical advice
Contact with skin:	Wash thoroughly with water.
Action on Spillage and Method of Disposal	
Spillage of contents should be taken up with absorbent material wetted with an appropriate disinfectant. Rinse area with an appropriate disinfectant followed by water. Absorbent materials used to treat spillage should be treated as biological waste.	

15. LIABILITY AND LOSS

In the event that this document is translated into another language, the English language version shall prevail in the event of any inconsistencies between the documents.

Unless expressly stated otherwise by NIBSC, NIBSC's Standard Terms and Conditions for the Supply of Materials (available at http://www.nibsc.org/About_Us/Terms_and_Conditions.aspx or upon request by the Recipient) ("Conditions") apply to the exclusion of all other terms and are hereby incorporated into this document by reference. The Recipient's attention is drawn in particular to the provisions of clause 11 of the Conditions.

16. INFORMATION FOR CUSTOMS USE ONLY

Country of origin for customs purposes*: United Kingdom * Defined as the country where the goods have been produced and/or sufficiently processed to be classed as originating from the country of supply, for example a change of state such as freeze-drying.
Net weight: 10g
Toxicity Statement: Toxicity not assessed
Veterinary certificate or other statement if applicable.
Attached: No

17. CERTIFICATE OF ANALYSIS

NIBSC does not provide a Certificate of Analysis for WHO Biological Reference Materials because they are internationally recognised primary reference materials fully described in the instructions for use. The reference materials are established according to the WHO Recommendations for the preparation, characterization and establishment of international and other biological reference standards http://www.who.int/bloodproducts/publications/TRS932Annex2_Inter_biologicalstandardsrev2004.pdf (revised 2004). They are officially endorsed by the WHO Expert Committee on Biological Standardization (ECBS) based on the report of the international collaborative study which established their suitability for the intended use.